
Get Started

Follow the instructions below before you start using GUNNPOD RX



Ensure you carefully follow all instructions given to you by your prescriber. If you are unsure or do not understand how to use your device, consult your doctor or pharmacist for guidance.

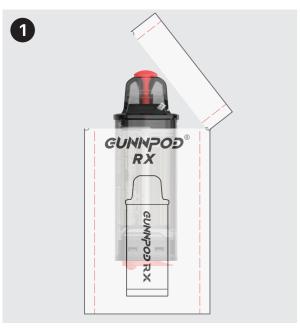

Remove Gunnpod RX pod from the sealed packaging




Remove the silicon seals from the mouthpiece and the bottom of the pod.



Insert pod into GPRX 650 Device




Place the mouthpiece against lips.

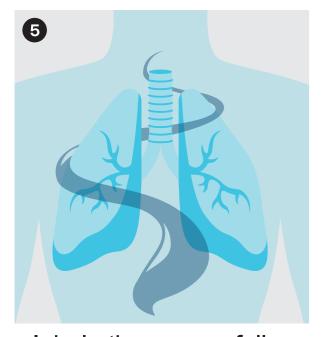

Inhale the vapour fully. (Do not swallow it)




Exhale

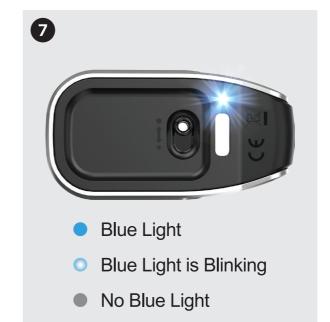

The blue light at the bottom of the device will light up when inhaling.

If the blue light is blinking, it indicates low battery.

If there is no blue light, the battery is empty. Charge now to resume use.

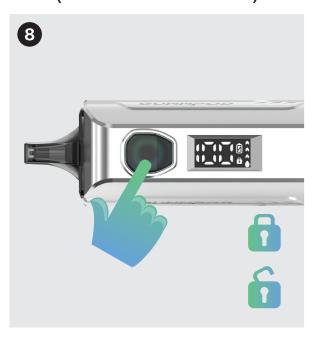

Tap the device button 3 times to activate the Child Lock system.


To deactivate, tap the button 3 times again.



You will know that
Gunnpod RX pod is empty
when there is insufficient
liquid in the pod.



Use a cotton swab with isopropyl alcohol to clean the mouthpiece and bottom air intake outlet and then dry it thoroughly. This will prevent any residue build-up.

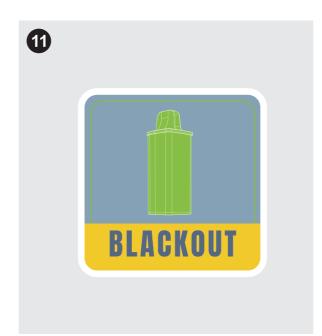

Keep Gunnpod RX pod sealed in its packaging until required for use.